Distributed by / Vertrieb / Distribuido por / Distribué par / Distribuito da / Gedistribueerd door / Distribuuje /  
Дистрибутор / Distribueret af / Διαμέτρεται από / Jakelija / Forgalmazó / 販売元 / Distribuert av / Dystrybutor /  
Distribuido por / Distributor / Дистрибутор / Distribútor / Distributer / Distribueraed av / Turustaja / Izplatītājs

**GB** B. Braun Medical Ltd.  
Sheffield, S35 2PW - United Kingdom

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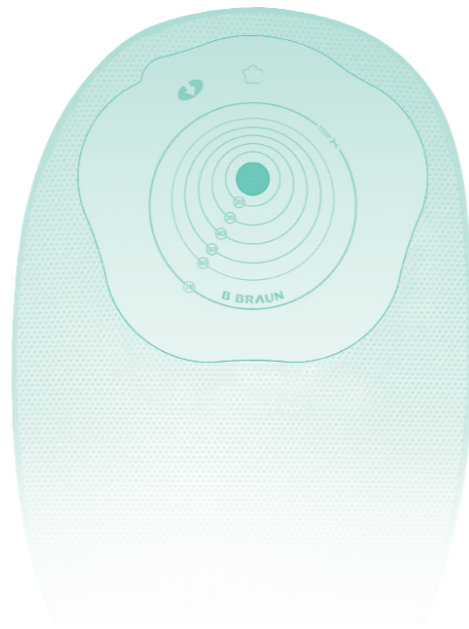
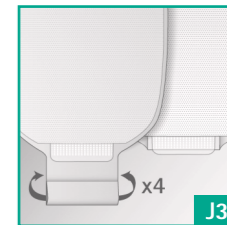
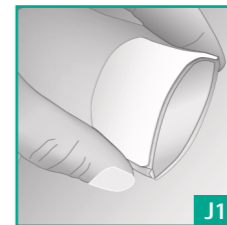
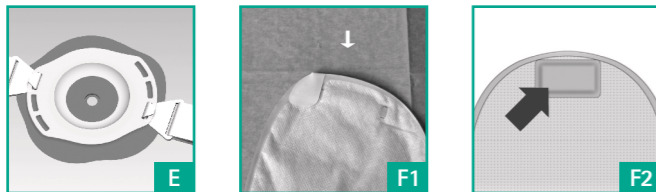
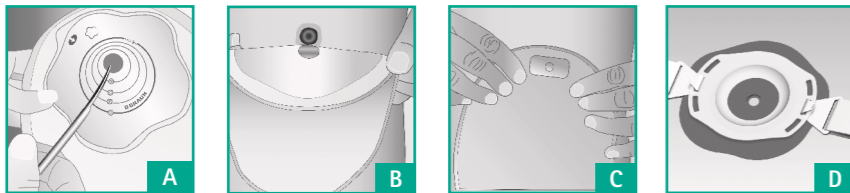
## Flexima® / Softima®

## EN Instructions for use

**What Flexima®-Softima® is and what is used for:**

Flexima®-Softima® are 1-piece collection pouches for enterostomates either:

- Closed for enterostomates (flat & convex).
- Drainable for colostomate with "Roll'Up" (flat & convex) outlet or a "High Flow" cap allowing connection to a B. Braun distal bag.

Flexima® / Softima® may be used concomitantly with B.Braun stoma care accessories.

**Intended purpose and indication:**

The product is indicated for collection and containment of stools by patients with colostomate. The product is not intended to be used on a skin or mucus membrane presenting a pathological change or change following disease or a wound.

Closed pouches are primarily intended for colostomates; drainable pouches for colostomates with soft or liquid stools and ileostomates; drainable "High Flow" pouches for ileostomates and jejunostomy (in case of high flow liquid stools).

**Clinical benefits & performances:** Ostomy appliances compensate for the loss of faecal continence.

**Patient population:** Colostomate adults or children above four years.

**Contraindication:** For the closed pouch Flexima® / Softima®, we do not recommend for use by colostomates with liquid stools.

**Residual risks or side effects:** Peristomal skin complications can occur while using Flexima® / Softima®.

**1 Preparing the skin**

- Wash your hands before starting the procedure.
- Gently clean the peristomal area with warm water. Avoid ether, alcohol or any other topical product which could interfere with the skin protector adhesion. Carefully dab dry the skin with soft and non-shedding cloths.

**2 Applying the pouch**

For drainable "Roll'Up" and "High Flow" pouches, close the closure outlet:

- Drainable pouches with integrated Roll'Up clamp\*: Fold the extremity of the sleeve four times using the guides and fold the tab back (J3). To seal the outlet, make sure that the two self-fastening strips are aligned, facing each other and apply pressure on all their length.
- Drainable "High Flow" pouches: Close the cap as far as maximum (K).

If you are using a Cut-To-Fit product, cut the central opening of the hydrocolloid adhesive to the size of your stoma (A) possibly with the help of the Stoma Guide. Do not cut beyond the maximum possible diameter and make sure that there is always a 2 to 3 mm margin around your stoma in order not to strangle or hurt the stomal mucosa.

## MD

- EN Medical device as per 2017-745/EU
- DE Medizinprodukt gemäß 2017-745/EU
- ES Producto sanitario según el Reglamento (UE) 2017-745
- FR Dispositif médical selon 2017-745/EU
- IT Dispositivo medico secondo 2017-745/UE
- NL Medisch hulpmiddel conform 2017-745/EU
- BG Медицинско изделие съгласно 2017-745/EU
- CS Zdravotnický prostředek dle nařízení (EU) 2017/745
- DA Medicinsk udstyr i henhold til 2017-745/EU
- FI Lääkinnällinen laite direktiivin 2017-745/EU mukaisesti
- EL Ιατροτεχνολογικό προϊόν σύμφωνα με τον Κανονισμό 2017/745/EE
- HU Orvosi eszköz, a 2017/745/EU rendeletnek megfelelően
- JA 2017-745/EUに從った医療機器
- NC Peranti perubatan berdasarkan 2017-745/EU
- PL Wyrób medyczny zgodnie z rozporządzeniem (UE) 2017/745
- PT Dispositivo médico de acordo com a diretiva 2017-745/EU
- RO Dispozitiv medical conform 2017-745/EU
- RU Медицинское изделие согласно 2017-745/EU
- SK Zdravotnícka pomôcka podľa normy 2017-745/EU
- SL Medicinski pripomoček v skladu z 2017-745/EU
- SR Medicinsko sredstvo u skladu sa 2017-745/EU
- SV Medicinteknisk produkt enligt 2017-745/EU
- ET Meditsiiniseade 2017-745/EU kohaselt
- LV Mediciniskā ierīce saskaņā ar 2017-745/ES

Separate the pouch walls by crumpling it slightly to introduce air into it.

Grasp the pouch by the sides (C).

Peel off the release from the skin protector and position the bottom of the flange opening around the base of the stoma (B). Press the bottom part of the skin protector smoothly onto the skin, then place and press onto the upper part taking care not to make folds.

In order to improve the adhesion, place hands over the skin protector to warm it up (C).

### ③ Emptying of drainable pouches

Position yourself appropriately.

■ Drainable "Roll'Up" pouches:

Release the two self-fastening strips and unroll the sleeve towards the toilet bowl. Be careful, emptying will start right at this point. To fully open the outlet, apply slight pressure with your fingers on guides (J1). Once the emptying is complete, expel, as much as possible, any matter still in the sleeve by pressing it lightly from top to bottom. Carefully clean its extremity avoiding the use of shedding cloths (J2).

Please note: the closed sleeve is hidden under the soft backing material (J3).

■ Drainable "High Flow" pouches:

Remove the cap by pulling it straight (K) and point it towards the toilet bowl. Once the emptying is complete, clean the cap.

After emptying of drainable pouches, close the closure outlet again as indicated above.

### ④ Using B. Braun belt

If you wish to use a B. Braun belt, choose two loops that are diametrically opposed (D-E).

### ⑤ Using filter protection stickers

Before bathing or showering, stick the filter protection sticker over the gas evacuation openings (F1) in order to maintain filter performances. For split cover, this must require to lift the comfort film and to stick the filter protection filter directly on the pouch films. After bath or shower, remove the filter protection stickers.

### ⑥ Removing the product

Pull slightly on the top of the skin protector and remove the pouch from the top to the bottom, while maintaining gentle pressure on the skin around the stoma with your other hand.

If you note any change on your stoma and/or peristomal skin, please contact your health care professional.

*\*Availability may vary from one country to another*

National regulations for the destruction of potentially infectious materials must be observed.

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