

Handling Instructions Sterican® MIX

Single-use needle for drug admixture




Carefully remove Sterican® MIX from the packaging.




Attach the needle securely to any Luer or Luer Lock syringe by twisting the syringe clockwise while holding the needle close to its base.



Remove the needle cap carefully straight from the needle hub.

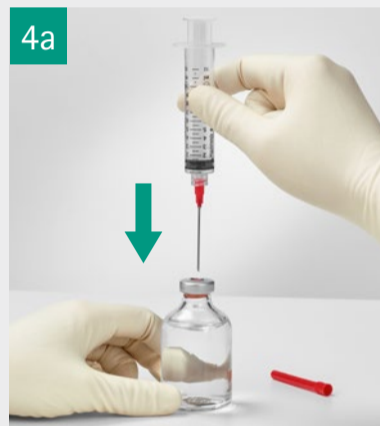


4a | For Sterican® MIX **semi-blunt needle**, pierce through the vial stopper in a 90° angle in order to reduce the possibility of coring to a minimum. Sterican® MIX semi-blunt needle is intended to puncture vials with a rubber stopper. It can be used as well with open containers.

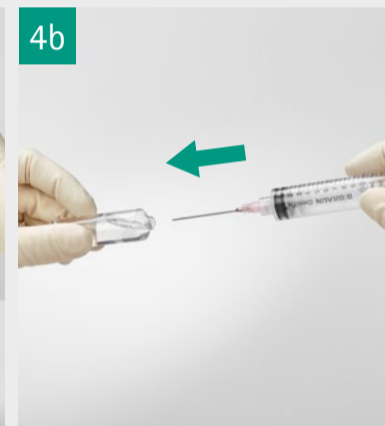

4b | For Sterican MIX® **blunt needle**, insert the needle into the opened glass ampoule. Sterican MIX® blunt needle should be used with glass ampoules or other open containers only.



Perform the admixture by drawing up required medication from the vial / glass ampoule.

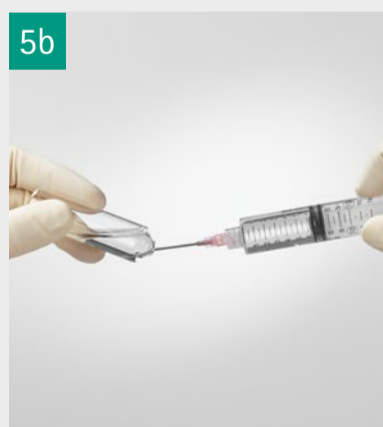

Discard Sterican® MIX immediately after application according to hospital policies and ensure a second needle is used for additional admixtures or hypodermic injections.

Note: Handling steps 1 – 3 and 6 are the same for both Sterican MIX® semi-blunt needle and Sterican MIX® blunt needle. For simplicity only Sterican MIX® semi-blunt needle is depicted in these handling images.