


AESCULAP® SQ.line® laparoscopic clip appliers

Disassembly – Assembly Instructions

Disassembly



■ To disassemble the SQ.line® clip applier, hold the handle on the black part.

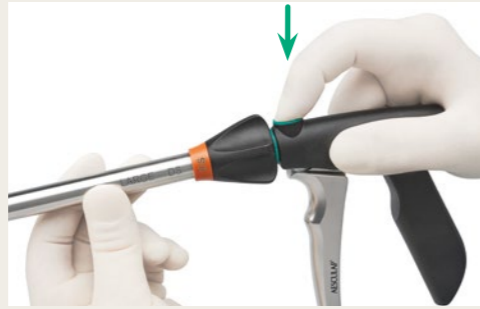

■ Push the black disassembly button on top of the handle until the stop.



■ While holding down the disassembly button, separate shaft and handle at the green separation ring.




■ Remove the push rod from the shaft.




■ Now the SQ.line® clip applier is fully disassembled.

Assembly

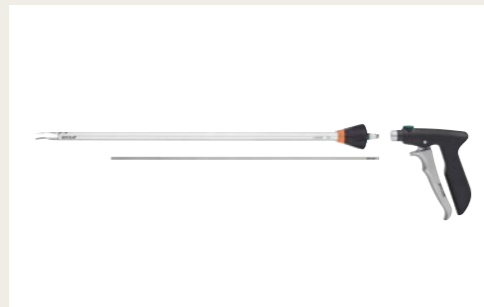

■ The SQ.line® laparoscopic clip applier consists of 3 components: shaft, push rod and handle.

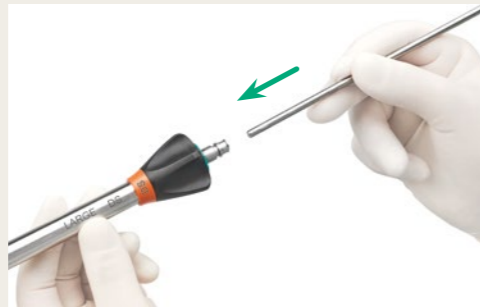

■ To assemble the clip applier, insert the push rod into the shaft up to the stop.
Note: The direction of insertion does not matter.



■ It is recommended to hold the shaft with the rotation knob slightly upwards to avoid the push rod sliding out.



■ Hold the handle of the clip applier on the black part, ensuring the mobility of the movable part.




■ Insert the shaft into the handle in one continuous movement until the shaft clicks in place.
Check proper assembly by pushing the movable part of the handle. The jaws of the applier will close simultaneously.



■ Now the SQ.line® clip applier is fully assembled.

Care

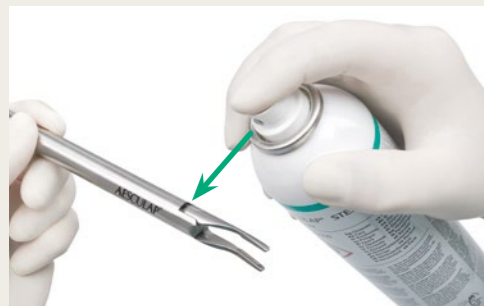


■ Lubricate the defined places on the clip applier with Aesculap® Sterilit® Oil after each cleaning cycle.



Processing & Accessories

- Reprocess the product immediately after use
- The product can be sterilized either in disassembled or in assembled condition
- For advice on cleaning, disinfection and sterilization, please follow the instructions for use
- For the latest information on reprocessing, please also refer to <https://eifu.bbraun.com>
- For accessories and spare parts, please refer to brochure C11702 or visit www.endoscopy-catalog.com

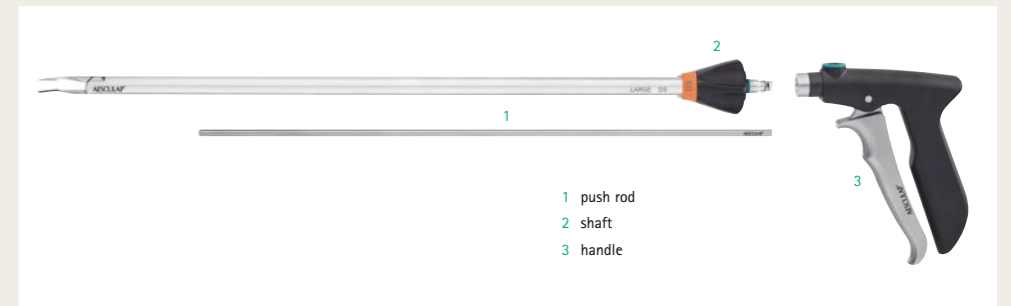

■ In order to ensure the preservation of full guarantee and warranty rights, only persons authorized by Aesculap may carry our repairs.
Please send products for repair to: Aesculap Technical Service, Am Aesculap-Platz, 78532 Tuttlingen, Germany. Additional service addresses may be obtained from the same address or via ats@aesculap.de